RAM SHELLAC BASE PLATE

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Product Description: - RAM SHELLAC base plate contains shellac which is a natural resin having low fusing properties and fillers for brittleness making it ideal for fabrication of trial denture base during making of Acrylic / Flexible dentures. It can also be used to fabricate custom impression trays for recording secondary impression in completely edentulous patients. Shellac being a low fusing natural resin is dimensionally stable at working room temperature and has a softening point between 50-60°C, making it comfortable for handling. It can be easily moulded or adapted to the desired shape by softening it uniformly, it retains the given shape and does not shrink or warp back on cooling.

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Composition: - Shellac, Fillers, Additives, Pigments & Excipients.

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Procedure of Use & Product Handling: a) Fabrication of Trial Denture base: Trial denture is fabricated using RAM SHELLAC base plate over the final impression model, to facilitate in recording the occlusal relation and teeth arrangement. 1) RAM SHELLAC base plate is heated directly on flame, to soften its consistency making it adaptable. 2) Softened plate is adapted on gypsum model with uniform pressure to maintain the thickness of the trial denture at 1-2 mm. 3) Excess shellac extending outside the model is cut by using a hot wax knife. 4) The border is moulded to maintain the thickness of the flanges, to give extra strength to the trial denture base.

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Instructions for Use: Fabrication of trial denture base and custom impression tray during prosthetic rehabilitation in edentulous patients.

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Storage Conditions: 1) Keep Away from Direct Sunlight 2) Keep Away from Moisture 3) Keep locked up and out of the reach of children. 4) Store at Temp. between 10°C to 30°C. 5) Shelf life not affected after opening if stored in a cool dry place.

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Disposal conditions: Dispose in accordance with local, state and federal environmental regulations.

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Warning: 1) Softened / Molten RAM SHELLAC base plate may cause burns to skin if it is overheated. 2) If the RAM SHELLAC base plate is heated above 100°C while heat source remains active, the plates may ignite and burn on their own. 3) Ensure device to be used only by a qualified dentist or dental technician. 4) Avoid inhalation of hot RAM SHELLAC base plate vapours. 5) RAM SHELLAC base plate should be inserted into the patient's mouth for a short time and not continuously.

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Precautions: 1) Avoid prolonged or repeated contact with skin (allergic contact dermatitis), avoid contact with eyes and inhalation of the fumes. Do not take internally. 2) Skin Contact: if hot material comes in contact with Skin, it may cause skin burn / sensitization, If contact with skin occurs immediately remove excess material and pour water on contacted part. If skin rash and sensitization or other allergic reaction occurs seek medical attention.

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Contraindications: It should not be used with patients who have a history of severe allergic reaction to any of the components.

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Interactions with other Dental Materials: No such Reactions Noted . Kindly communicate and inform manufacturer in details of any instance.

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Adverse Reactions: No such Reactions Noted . Kindly communicate and inform manufacturer in details of any instance.

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Presentation: Refer to outer packaging.

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Color & Thickness: Refer to outer packaging.

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☰ Symbol for "BATCH CODE" ☰ Symbol for "USE BY" ⚠ Symbol for "NON STERILE" ☀ Symbol for "KEEP DRY" ☀ Symbol for "KEEP AWAY FROM SUNLIGHT" 👶 Symbol for "KEEP OUT OF THE REACH OF CHILDREN" 🏢 Symbol for "MANUFACTURER NAME & ADDRESS" 📖 Symbol for "CONSULT INSTRUCTION FOR USE" 📄 Symbol for "CATALOGUE NUMBER" 🚫 Symbol for "DO NOT USE IF PACKAGE IS DAMAGED" 🌡 Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.