ZEUGENOL

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Product Details: - ZEUGENOL is an impression material which is the combination of zinc oxide and Eugenol (contained cloves'oil). An acid-base reaction takes place with the formation of zinc eugenolate chelate. Paste is very easy to mix, having right flow characteristics, dimensionally stable with appropriate setting time.

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Application :- Used for taking impressions of toothless jaws in individual impression trays for making removable full dentures, to enable mal-fitting dentures to be relined and also for temporary fixations of crowns, bridges and pin teeth.

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Composition:- Base paste: Zinc Oxide, petroleum jelly, Resin & Excipients. Catalyst Paste: Eugenol, petroleum jelly, Color & Excipients.

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Disclaimer:- The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instruction of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.

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Use & Seek Medical Attention: It should not be used with patients who have a history of severe allergic reaction to any of the components.

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Warning: Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

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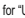
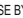
Presentation : Ref. ZEUGENOL 1 x 120 g, Base, 1 x 100 g Catalyst, Plastic Tube, 1- Mixing pad.

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Shelf Life: 3 Years from date of manufacturing.

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 Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"  Symbol for "KEEP AWAY FROM SUNLIGHT"  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"  Symbol for "DO NOT USE IF PACKAGE IS DAMAGED"  Symbol for "TEMPERATURE LIMITATION"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.